Aesculap[®] Dental Prosthetics

Competence in Dentistry

Aesculap Dental

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Crown widening pliers,
140 mm,
for removing crowns and
bridges.

*At the beginning
the crown is parallelly slit
with a relatively thin
instrument.*

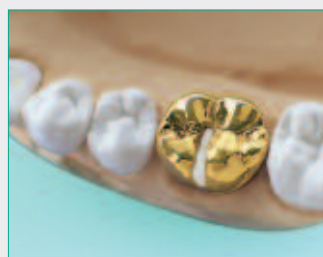
*To produce parallel slit edges
which can be easily gripped
with the pliers, a cylindrical in-
strument with diam. of approx
1.0 mm is guided perpendicular
to the tooth center line.*

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The teeth of the crown widening pliers are positioned in order to spread the crown.

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Finally the crown will be removed in the conventional way.

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Our tip:

The slit walls must be perpendicular. Always cut through the bottom crown edge and, if possible, extend the slit up to the masticatory surface. In the case of bridges, move the slits approximately 1/3 towards the suspension element.

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BÖHM telescope pliers firmly grips the inner crown so that the outer one can be separated easily. To avoid damage to the crown due to a stern pressure, the set screw can be adjusted.

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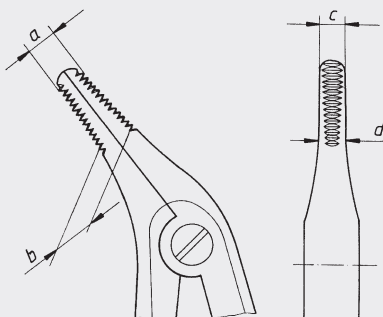
slender pattern,
the diamond powder
coated gripping
surfaces ensure an
especially firm grip

BÖHM	a	b	c	d
DP739R	3,3	4,0	2,3	3,0 mm
DP740R	4,4	5,0	3,4	3,8 mm
DP741R	3,3	4,0	2,3	3,0 mm

Provisorien- und Kronenabnehmezange Copper ring, temporary measure and crown removing pliers

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Forceps for removing copper rings, as well as temporary and permanent crowns. One jaw has a serrated surface, which allows for a sure grip on opened crowns: When the jaws meet at the crown, the serrations on the rounded jaw prevent the pointed jaw from slipping off and causing damage to the crown.

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Copper ring,
 temporary measure
 and crown removing pliers

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Crown remover

Crown- and bridge re-
mover, complete, with
3 points
Set consists of
DP781R, DP783R,
DP784R, DP785R

Crown- and bridge
remover,
Handle only

Points for crown- and bridge remover
DP781R

Kupferingabnehmerzangen, Messinstrumente
Copper Ring Removing Pliers, Measuring Instruments

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Steel ruler, stainless, graduated in millimeters and inches

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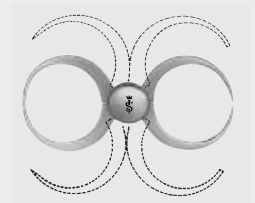
This invention is an instrument used in relation to CEREC Inlay/Veneer manufactured by the CEREC CAD-CAM machine. They are usable for upper and lower jaw insertions. By using finger slots one hand use is possible.

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Occlusal stop and 1 x 1 lateral tips provide a very secure grip.

Sharp pointed claws provide easy removal of well-fitted or tight-fitted inlays/veneers inserted at trial stage.

Finger Slot **DP803R** (to be ordered separately)

- ◆ for one hand use
- ◆ interchangeable
- ◆ rotating
- ◆ 3 sizes: 19, 21, 23 mm diam.

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Designed to minimize the risk of dropping inlays and veneers.

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Occlusal stopper, flexible working tips and 2 x 2 lateral claws provide a very secure grip for handling all different shapes and sizes of the inlay to minimize the risk dropping.

Wheel knob
– forward and backward rotation for precision opening and closing of the claws, to create hard/soft/release grip

NG
DP801R

150 mm, 6"

Gripping HOLLINSER
Fixierpinzette,
ohne Ring

Gripping HOLLINSER,
fixation forceps,
without finger ring

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Finger rings in 3 sizes,
for DP800R and
DP801R

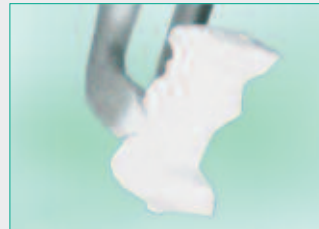
I Anpassen (Gripping HOLLINSER, DP801) Trial Fitting (Gripping HOLLINSER, DP801)

1. Inlay entgraten.
Margin Trimming

2. Probeweise
Anpassung
Trial fitting
for path of
insertion

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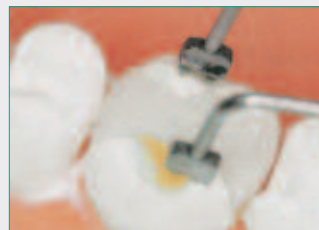
JESCO®-FORM impression trays and impression trays acc. WÖSTMANN with perforated pattern. Using plastic or resilient impression compounds, these materials will penetrate the holes and provide the necessary support for a correct impression. The impression trays are made of stainless steel 18/8.

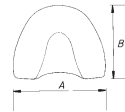
The impression trays for lower jaw acc. WÖSTMANN in addition have the following characteristics and advantages:

- ◆ Modified tray shield with changed tray curve.
- ◆ Lengthened tray with simultaneous reduction of transversal expansion.
- ◆ All teeth are seized, incl. the third molars
- ◆ Sufficient and uniform thickness of the impression compounds in the area of lingual undercut of the teeth.

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* The shapes of the impression trays are based on an analysis of naturally toothed patients of:

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according to WÖSTMANN
Impression trays for toothed
lower jaws, perforated pattern

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JESCO®-FORM Impression Trays
for toothed lower jaws

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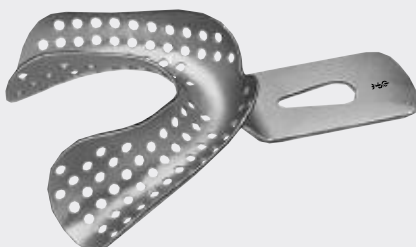
JESCO®-FORM Impression Trays
for edentulous lower jaws

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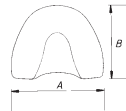
JESCO®-FORM Impression Trays
for partly toothed lower jaws

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A B
UK 00 55 x 37 mm

JESCO®-FORM Impression Trays
for children's lower jaws

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
A B
OK 0 59 x 42 mm

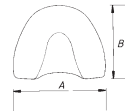JESCO®-FORM Impression Trays
for children's upper jaws

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JESCO®-FORM Impression Trays
for toothed upper jaws

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JESCO®-FORM Impression Trays
for narrow toothed upper jaws

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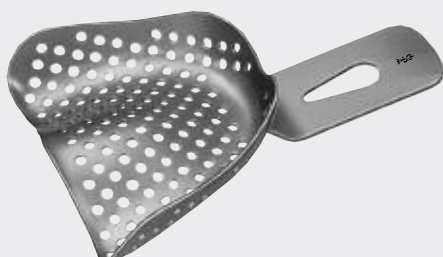
JESCO®-FORM Impression Trays
for partly toothed upper jaws

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
for upper jaw left resp. lower
jaw right

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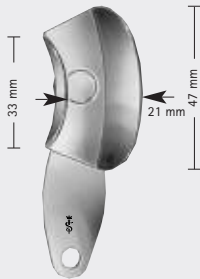
for upper jaw right resp. lower
jaw left

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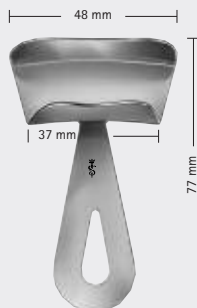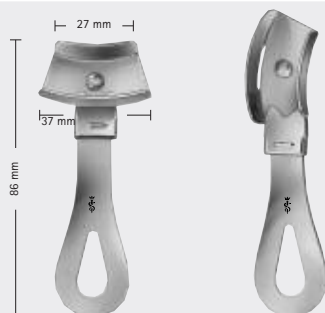
The kaiser pattern occlusion impression tray is used for thermoplastic and rubber elastic impression compounds as well as for alginates. Thanks to its special shape, the impression tray carries the impression compound right into those parts of the jaw which are to be moulded, thus ensuring an impression as accurate as possible.

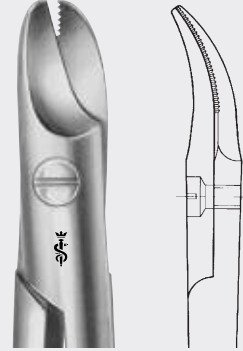
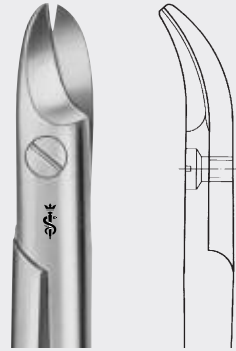
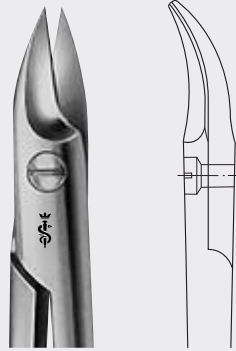
- ◆ Simple, quick impression with total occlusion
- ◆ Positive guiding of the impression compound due to sloping surfaces
- ◆ Excellent retention of the compound in the tray with no danger of deformation when the impression is removed from the patient's mouth.
- ◆ The plain surfaces make it easy to clean the tray thoroughly

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The Kaiser pattern occlusion impression tray, with slots for insertion of paper stripes, suitable for left and right side

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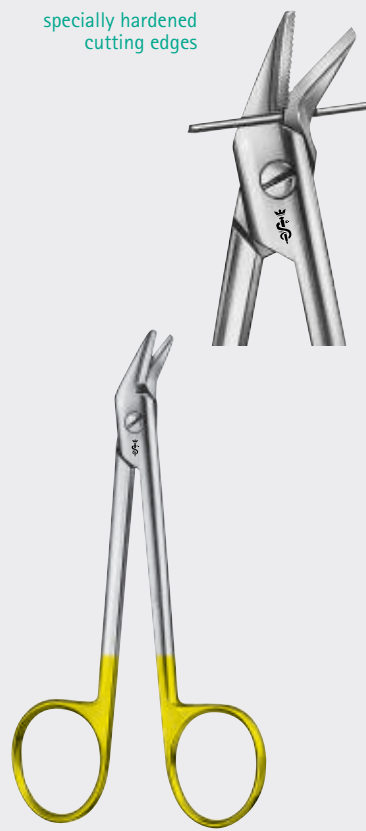
one blade serrated

for soft wire up to 1.2 mm
 for hard wire up to 0.6 mm

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one blade serrated

for soft wire up to 1.2 mm
 for hard wire up to 0.8 mm

MERSHON
DP756R
 140 mm, 5¹/₂"

Kronenandrücker

Crown depressor