## Superfiller 60g

|           |  |
|-----------|--|
| <b>en</b> | Protective skin barrier paste                                  |
| <b>de</b> | Hautschutzpaste  |
| <b>es</b> | Pasta protectora para piel periestomal                         |
| <b>fr</b> | Pâte de protection péristomiale                                |
| <b>it</b> | Pasta protettiva per la cute periestomale                      |
| <b>nl</b> | Huidbeschermende pasta   |
| <b>bg</b> | Паста за създаване на защитен слой върху кожата                |
| <b>cs</b> | Ochranná pasta   |
| <b>da</b> | Beskyttende hudbarriere pasta                                  |
| <b>el</b> | Αλοφική προστατευτική δερματική φραγμού                        |
| <b>fi</b> | Ihonsuojapasta   |
| <b>hu</b> | Bőrvédő paszta   |
| <b>lv</b> | Ādu aizsargājoša pasta   |
| <b>pl</b> | Pasta uszczelniająco-gojąca                                    |
| <b>pt</b> | Pasta protetora de barreira cutânea                            |
| <b>ro</b> | Pastă cu rol protector la nivelul pielii                       |
| <b>ru</b> | Паста-герметик для защиты и выравнивания кожи в перистомальной |
| <b>sk</b> | oblastiPasta na ochranu kože                                   |
| <b>sl</b> | Pasta za zaščito kože  |
| <b>sr</b> | Zaštitna zaptivna krema za kožu                                |
| <b>tr</b> | Koruyucu cilt bariyeri macunu                                  |
| <b>zh</b> | 护肤隔膏膏  |

### INSTRUCTIONS FOR USE

1. PRODUCT DESCRIPTION
Protective skin barrier paste, Peristomal Paste is a convenient and effective skin barrier paste, for use with colostomies, ileostomies, urostomies and fistulas, it can also be used to fill in spaces caused by skin folds, lesions or other irregularities of the skin.

2. INTENDED PURPOSE
Peristomal Paste is intended for use as a skin barrier paste, to fill any skin surfaces that affect the ability to securely attach an ostomy appliance.

2.1 INTENDED POPULATION
The product may be used on individuals with any of the aforementioned conditions.

2.2 INTENDED USER
The intended users are health care professionals, caregivers and patients in a hospital or home environment.

3. INDICATIONS
Peristomal Paste is indicated as a skin barrier paste for use with colostomies, ileostomies, urostomies and fistulas. It is used to fill any irregularities on the skin surface that affect the ability to securely attach an ostomy appliance.

4. WARNINGS

- Do not use if the tube is damaged or open before use.
- For external use only.
- Do not apply to cuts or open wounds or damaged skin.
- Contains alcohol.
- May sting slightly when applied.
- Should skin irritation occur, contact your doctor or another health professional.

5. INFORMATION FOR USE
Method of Application:
Peristomal Paste can be applied with a moistened finger around the stoma, filling in the spaces caused by skin folds, lesions or other irregularities of the skin. After having adapted the base plate or flange to the size and shape of the stoma, place the base plate directly on the paste that has been applied to the skin. Press the wafer down around the stoma in such a way that the paste is distributed under the wafer.

Method of Removal:
The paste is easily removed 48 hours after application. If removing the paste before this time, use warm water to remove it. It is not necessary to remove all the paste from the skin prior to a further application. The residue paste may be left in order to avoid possible irritation and the new paste applied over this residue. After use, close the tube securely.

Composition:
Contains Butyl monoester of PVM/MA

6. CLINICAL BENEFIT / SPECIAL NOTES
Prevents leakages that cause skin irritations by creating an effective seal for increased security. The paste is easy to spread and is ideal for uneven skin surfaces.

7. STORAGE
Store dressings at ambient temperature and humidity, away from direct sunlight.

8. DISPOSAL
Can be disposed based on the hospital or healthcare professional advice.

9. PRESENTATION
A 60g aluminium tube, presented in a unit box (1 tube per box) and packed with an IFU.

10. DISTRIBUTOR
B. Braun Hospicare Ltd. Collooney, Co. Sligo, F91 C892, Ireland
Phone: +353 71 911 5000
Fax: +353 71 911 5064
www.bbraun.com/living-with-a-stoma

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B. Braun Hospicare Ltd. Collooney, Co. Sligo, F91 C892, Irsko
8. Braun Hospicare Ltd. Collooney, Co. Sligo, F91 C892, Irsko